Test FOB

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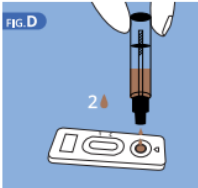
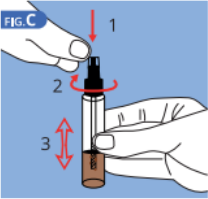
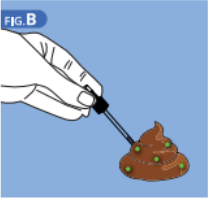
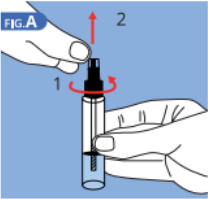
1. FOB

Many diseases may result in hidden blood in the feces. This is known as fecal occult blood (FOB), human occult blood, or human hemoglobin. In the early stages, gastrointestinal problems such as colon cancer, ulcers, polyps, colitis, diverticulitis, and fissures may not show any vFecal occult blood tests are used as a screening tool for detecting lower gastrointestinal (GI) bleeding that may be related to iron deficiency anemia, peptic ulcer, ulcerative colitis, polyps, and colorectal cancer. The Fecal Occult Blood Rapid Test is recommended for use by health professionals in routine physical examinations and in monitoring for GI bleeding in patients in hospitals or in physicians’ offices visible symptoms, only occult blood.

2.SAMPLE COLLECTION AND PREPARATION

small amounts of stool.

3. CONTENTS

The box contains an HP kit including:
1 hermetically sealed protective aluminum pouch, containing:

- 1 test cassette,
- 1 dropper bottle containing 1 ml of diluent,
- 1 disinfectant swab,
- 1 desiccant sachet,
- 1 instructions for use leaflet.

4. PRECAUTIONS FOR USE

- For professional use only (in vitro diagnostic use).
- Do not use after the expiration date.
- Keep the test in its sealed pouch until use.
- All samples should be considered potentially infectious and handled with care.
- After use, dispose of the test according to local regulations (biological waste).

5. STORAGE AND STABILITY

Store in the original packaging at room temperature or refrigerated (2–30°C). The test is stable until the expiration date printed on the sealed pouch or box label. The test must remain in the sealed pouch until use.

DO NOT FREEZE. Do not use beyond the expiration date.

6. QUALITY CONTROL

The test includes an internal control system represented by the colored line appearing in the control region (C). This line confirms that the test has been performed correctly with a sufficient sample volume.

The absence of a control line indicates that the test has not functioned properly.

Test FOB

7. PROCEDURE

- 1.Wash your hands thoroughly with soap, then rinse with clean water.
2. Tear open the protective pouch and remove **only** the test strip (cassette) and the dropper.
3. Collect a random stool sample in a clean container. Open the cap of the collection tube and remove the sample collection stick
3. Collect small amounts of stool from **at least five different locations**. Do **not** collect a large mass of stool, as this may invalidate the test result
- 4.Return the collection stick to the tube and close it tightly. The sample is now ready for testing, transport, or storage
5. When ready to perform the test, open the pouch at the designated tear notch and remove the test strip. Place it on a clean, flat surface.
6. Shake the sample collection tube vigorously to ensure proper suspension of the specimen
- 7.Test preparation:** Hold the tube upright and break off the designated tip.
- 8.Add **two drops** of the suspension into the test well.
- Do not add excessive sample.**
- 9.The result can be read within **15 minutes**.
- Add two drops of the suspension into the test well.
- Do not add excessive sample.
- 10.The result can be read within 15 minutes.

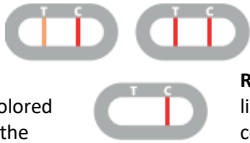
8. INTERPRETATION OF RESULTS

READ THE RESULTS WITHIN 10 MINUTES. DO NOT INTERPRET RESULTS AFTER 10 MINUTES.

The intensity of the line color does not affect the interpretation of the test result.

POSITIVE RESULT

Two colored lines appear: one in the test region (T) and one in the control region (C).

NEGATIVE

Only one colored line appears in the region (C).

INVALID RESULT

No line appears, or a colored line appears in the test region (T) without a line appearing in the control region (C).

In this case, the test cannot be interpreted and must be considered invalid. It is recommended to repeat the test using a new cassette.

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