Test Vitamine D

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1. Vitamin D TEST

Vitamin D refers to a group of fat-soluble secosteroids responsible for increasing the intestinal absorption of calcium, iron, magnesium, phosphates, and zinc. In humans, the most important compounds in this group are vitamin D3 and vitamin D2. Vitamin D3 is naturally produced in human skin through exposure to ultraviolet light, while vitamin D2 is mainly obtained from dietary sources. Vitamin D is transported to the liver, where it is metabolized into 25-hydroxyvitamin D.

2. SAMPLE COLLECTION AND PREPARATION

One drop of blood.

3. CONTENTS

The box contains an Vitamin D test kit including:

- 1 hermeticallysealed protective aluminumpouchcontaining:
 - 1 test cassette,
 - 1 plastic pipette,
 - 1 dropper bottlecontaining 1 ml of diluent,
 - 1 sterilelancet for bloodsampling,
 - 1 disinfectantswab,
 - 1 desiccant sachet,
 - 1 instruction leaflet.

4. PRECAUTIONS FOR USE

- For professional use only (in vitro diagnostic use).
- Do not use after the expiration date.
- Keep the test in itssealedpouchuntil use.
- All sampleshouldbeconsideredpotentiallyinfectious and handledwith care.
- After use, dispose of the test according to local regulations (biologicalwaste).

5. STORAGE AND STABILITY

Store in the original packaging at room temperature or refrigerated (2–30°C). The test is stable until the expiration date printed on the sealedpouch or box label. The test must remain in the sealedpouchuntil use.

DO NOT FREEZE. Do not use beyond the expiration date.

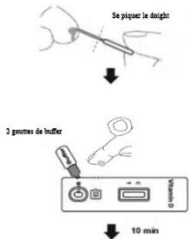
6. QUALITY CONTROL

The test includes an internal control system represented by the colored line appearing in the control region (C). This line confirmsthat the test has been performedcorrectlywith a sufficientssample volume.

The absence of a control line indicatesthat the test has not functionedproperly.

7. PROCEDURE

- 1.Wash hands with soap and rinsewith clean water.
- 2.Tear open the protective pouch and removeonly the test cassette and pipette.

Test Vitamine D

- 3.Remove and discard the protective cap from the lancet.
- 4.Press the exposed part of the lancet firmly against the fingertip.
- 5.Keeping the hand facing down ward, massage the punctured finger tip to obtain a well-formed drop of blood.
- 6.Without squeezing the pipette bulb, place the pipette in contact with the blood drop. You may need to massage the finger again to obtain more blood if the required levels not reached.
- Avoid air bubbles.
- 7.Deposit the collecte dblood into the sample well of the cassette.
- 8.Wait until the blood has completely flowed into the well.
- 9.Add 2 drops of diluent solution into the samplewell of the cassette.
- 10.Waitbetween 5 and 10 minutes to read the results.

8. INTERPRETATION OF RESULTS

READ THE RESULTS WITHIN 10 MINUTES. DO NOT INTERPRET RESULTS AFTER 10 MINUTES.

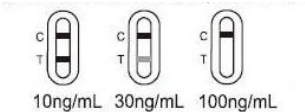
The intensity of the line color does not affect the interpretation of the test result.

POSITIVE RESULT

Deficient (0–10 ng): The intensity of the line in the Test (T) region is equal to or darker than the line in the Control (C) region.

Insufficient (10–30 ng): The intensity of the line in the Test (T) region is darker than the line in the Control (C) region.

Sufficient (30–100 ng): The intensity of the line in the Control (C) region is very faint compared to the line in the Test (T) region.

INVALID RESULT

No line appears, or a colored band forms under marker 1 and/or T without a line appearing under the Control (C) marker.

In this case, the test cannot be interpreted and must be considered invalid.

It is recommended to repeat the test using a new cassette.

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